







BD Vacutainer® UltraTouch™ Push Button Blood Collection Set

Enabling improved blood culture collection to aid diagnostic accuracy



The facts about sepsis

Sepsis is a life threatening condition caused by the body's response to an infection, resulting in organ dysfunction and failure.¹

			
Globally, an estimated 31.5 million cases of sepsis are diagnosed each year, with an estimated 5.3 million deaths . ²	As a leading cause of mortality and critical illness worldwide, sepsis hospital mortality rates range from 25%–30% . ²	Clinical studies have demonstrated a two-fold increase in mortality caused by sepsis when inappropriate antimicrobial therapy is given. ³	Sepsis has the highest cost per stay at \$18k which accounts for 5.2% of national U.S. costs and is the most expensive condition to treat. ⁴


Blood culture is critical for detecting and treating sepsis⁵

The pathway to the right diagnosis begins with the blood culture sample collection

Challenges in patient preparation and sample collection can result in preanalytical errors, accounting for up to 62% of all errors made in routine chemistry laboratory diagnosis.⁶ These errors may also occur in blood culture sample collection as the methods of blood collection are similar.

Best practice in sample collection aims to minimise contamination and ensures that sufficient blood volume is collected. If contamination occurs or insufficient blood is collected, it may be impossible to accurately recover the organism(s) causing infection and may affect diagnosis and treatment of the patient.


Contamination-free, total blood volume, and time to incubation play a vital role in accurate diagnostics based on best practices below*:

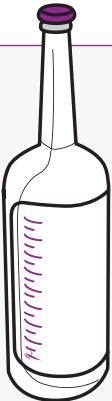

Over-filling leads to false positives⁷

8 to 10mL ✓

Under-filling leads to low detection of bacteria⁸



x3

Acquiring at least 3 sets of BD BACTEC™ vials can improve organism recovery to 98%^{9,10}

<4 hours

blood culture bottles must be incubated within 4 hours of blood collection⁸


* best practices are based on adult volumes. Please refer to your local guidelines when collecting blood from younger and critically anaemic patients

Advances in technology can help improve blood volume collection

Sufficient blood volume is one of the critical determinants for an optimal yield of organisms. The yield of pathogens increases in direct proportion to the volume of blood cultured.¹¹

The BD Vacutainer® UltraTouch™ Push Button Blood Collection Set incorporates BD RightGauge™ Ultra-Thin Wall Cannula Technology, which:

1. Allows greater sample fill volume to be collected in blood culture bottles¹²
2. Improves blood flow rates¹²
3. Provides a larger inner diameter while maintaining a set outer diameter and depending on gauge size can provide various levels of patient comfort¹³

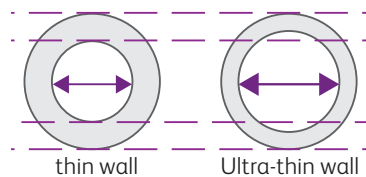

BD Vacutainer® UltraTouch™ Push Button Blood Collection Set can show improvements in blood volume and collection time

In a study, the 25 gauge BD UltraTouch™ has an average flow rate of 0.2195 mL/sec¹². Using the recommended volume required for a blood culture bottle, 10mL, the total time for collection can be calculated as given below to enable comparison with another device:

- $10\text{mL} \div 0.2195 \text{ mL/sec} = 46 \text{ seconds}$ per bottle using BD UltraTouch™. In comparison with BD PushButton™ the total time is **98 seconds** per bottle. This means that BD UltraTouch™ can be **52 seconds faster** per blood culture bottle. This can make a significant difference on smaller and difficult veins.
- For the recommended total collection volume of 40 to 60mL of blood, there can be a **saving of ~3.5 to 5.2 minutes** per blood collection.



We can help you implement best practices in the preanalytical phase of blood culture sample collection

At BD we offer valuable support with BD Professional Services when driving best practice to ensure high quality sample collection. We assess your sample management workflow to ensure all factors that contribute to sample quality are controlled and monitored. Please reach out to a local representative for a general consultation and a free Quick Reference Guide for blood culture sample collection.

BD Vacutainer® UltraTouch™ Push Button Blood Collection Sets:

Cat no.	Description	Shelf pack / case
368689	21G / 0.8mm x 19mm (0.75") needle with 305mm (12") tubing with pre-attached holder	20/100
368688	23G / 0.6mm x 19mm (0.75") needle with 305mm (12") tubing with pre-attached holder	20/100
368687	25G / 0.5mm x 19mm (0.75") needle with 305mm (12") tubing with pre-attached holder	20/100
367365	21G / 0.8mm x 19mm (0.75") needle with 305mm (12") tubing and luer adapter	50/200
367364	23G / 0.6mm x 19mm (0.75") needle with 305mm (12") tubing and luer adapter	50/200
367363	25G / 0.5mm x 19mm (0.75") needle with 305mm (12") tubing and luer adapter	50/200
364815	BD Vacutainer® One Use Holder	250/1,000

BD recommends for all needles and blood collections sets without a pre-attached holder that a BD Vacutainer® One-Use Holder be used

Optimise your blood culture sample collection and diagnostic process further with BD Professional Services, BD BACTEC™, and BD Synapsys™

Learn more about BD's commitment to Patient Safety
eu.bd.com/patient-safety



References

1. Singer M, Deutschman CS, Seymour CW, et al. The third International consensus definitions for sepsis and septic shock (Sepsis-3). JAMA. 2016;315(8):801-810.
2. Fleischmann C, Sheerag A, Adhikari NK, et al. Assessment of global incidence and mortality of hospital-treated sepsis. Current estimates and limitations. Am J Respir Crit Care Med. 2016;193(3):259-272.
3. LaRosa SP. Sepsis. www.clevelandclinicmeded.com/medicalpubs/diseasemanagement/infectious-disease/sepsis. August 2010. Accessed August 23, 2017.
4. Torio M, Moore B. National Inpatient Hospital Costs: The Most Expensive Conditions by Payer, 2013. HCUP Statistical Brief #204. Rockville, MD: Agency for Healthcare Research and Quality; May 2016.
5. Patel M. Utility of blood culture in sepsis diagnostics. J Acad Clin Microbiol. 2016;18(2):74-79.
6. Carraro P, Plebani M. Errors in a stat laboratory: types and frequencies 10 years later. Clin Chem. 2007;53(7):1338-1342.
7. Thompson F, Madeo M. Blood cultures: towards zero false positives. J Infection Prevention VOL. 10 SUPPLEMENT 1 SEPTEMBER 2009
8. Italian Consensus document, Execution, transport and preservation procedures for sampling for blood culture in cases of suspected sepsis, English version, 2018
9. Clinical and Laboratory Standards Institute m47-A Principles and procedures for blood cultures: Approved Guideline. 2007, 27:1, p5
10. Lee A, Mirrett S, Reller B, Weinstein M. Detection of Bloodstream Infections in Adults How Many Blood Cultures Are Needed. J Clin Microbiol, 2007, 45:11, 3546-3548
11. Lamy B, Dargère S, Arendrup MC, Parienti J-J, Tattevin P. How to Optimize the Use of Blood Cultures for the Diagnosis of Bloodstream Infections? A State-of-the Art. Front. Microbiol. 2016; 7:697.
12. Evaluation of Draw Volume, Fill Time and Fill Rate for BD BACTEC™ Blood Culture Bottles using the BD Vacutainer® UltraTouch™ Push Button Blood Collection Set compared to the BD Vacutainer® Push Button Blood Collection Set. White paper VS9381. Franklin Lakes, NJ: BD; 2019.
13. Mouser A, Uettwiller-Geiger D, Plokhoy E, Berube J, Ahuja AJ, Stankovic AK. Evaluation of pain and specimen quality by use of a novel 25-gauge blood collection set with ultra-thin wall cannula and 5-bevel tip design. J Appl Lab Med. 2017;2(2):201-210.

BD, Terre-Bonne Park - A4, Route de Crassier 17, 1262 Eysins, Switzerland



CE 2797 applies to BACTEC™. CE 0050 applies to BD UltraTouch™

bd.com

BD, the BD Logo, BACTEC, RightGauge, Synapsys, UltraTouch and Vacutainer are trademarks of Becton, Dickinson and Company or its affiliates. © 2022 BD. All rights reserved. BD-57297

